

SURGEONS' CHOICE Medical Compression Hosiery - Size Chart

Thigh High Open Toe Support Stockings with Graduated Compression

Parts/Inches	A	B	C	D	E
Ankle Circumference	6.5" - 8"	8" - 9.5"	9.5" - 11"	11" - 12.5"	12.5" - 14"
Calf Circumference	10.5" - 14.5"	11.5" - 15.5"	12.5" - 17"	13.5" - 17.5"	14.5" - 19.5"
Thigh Circumference	15.5" - 22"	17.5" - 24"	19.5" - 26"	22" - 28"	26" - 32"
Thigh Length	- 29"	- 30"	- 31"	- 32"	- 33"

Parts/cm	A	B	C	D	E
Ankle Circumference	16.5 cm - 20.5 cm	20.5 cm - 24.5 cm	24.5 cm - 28 cm	28 cm - 32 cm	32 cm - 35.5 cm
Calf Circumference	26.5 cm - 37 cm	29 cm - 39.5 cm	32 cm - 43 cm	34.5 cm - 44.5 cm	37 cm - 49.5 cm
Thigh Circumference	40 cm - 56 cm	44.5 cm - 61 cm	49.5 cm - 66 cm	56 cm - 71 cm	66 cm - 81 cm
Thigh Length	- 74 cm	- 76 cm	- 78 cm	- 81 cm	- 84 cm

Knee High Support Stockings with Graduated Compression

Parts/Inches	A	B	C	D	E
Ankle Circumference	6.5" - 8"	8" - 9.5"	9.5" - 11"	11" - 12.5"	12.5" - 14"
Calf Circumference	10.5" - 14.5"	11.5" - 15.5"	12.5" - 17"	13.5" - 17.5"	14.5" - 19.5"
Calf Length	- 16"	- 17"	- 18"	- 18"	- 18"
Foot Length	- 8.7"	- 8.9"	- 9.3"	- 10"	- 11"

Parts/cm	A	B	C	D	E
Ankle Circumference	16.5 cm - 20.5 cm	20.5 cm - 24.5 cm	24.5 cm - 28 cm	28 cm - 32 cm	32 cm - 35.5 cm
Calf Circumference	26.5 cm - 37 cm	29 cm - 39.5 cm	32 cm - 43 cm	34.5 cm - 44.5 cm	37 cm - 49.5 cm
Calf Length	- 41 cm	- 43 cm	- 46 cm	- 46 cm	- 46 cm
Foot Length	- 22 cm	- 22.5 cm	- 23.5 cm	- 25.5 cm	- 28 cm

Instructions for Measuring

Measurements should be taken following these steps as soon as possible after waking when swelling is at its lowest.

THIGH HIGH

1. Measure the ankle circumference at the narrowest part of the ankle above the ankle bone. Use the sizing charts to determine the appropriate size.
2. Measure the calf circumference at the fullest part of the calf.
3. Measure the thigh at just below the groin.
4. Determine the thigh length by measuring from the floor to the gluteal fold.

KNEE HIGH

1. Measure the ankle circumference at the narrowest part of the ankle above the ankle bone. Use the sizing charts to determine the appropriate size.
2. Measure the calf circumference at the fullest part of the calf.
3. Determine the calf length by measuring from the floor to the bend of the knee.
4. Determine the foot length by measuring from the longest toe to the heel.

